scout 5.7

Fitting Software



HANSATON scout v5.7
Installation Guide / Installations-Anleitung / Guide d'installation / Guía de instalación

Table of contents / Inhalt / Table des matières / Índice

English.....	2
System Requirements	3
Deutsch.....	4
Empfohlenes System.....	5
Français.....	6
Systèmes recommandés.....	7
Español	8
Sistemas recomendados	9

Installing HANSATON scout Software

1. Download and extract the installation file.
 - If the installation program does not start automatically, double click on setup.exe in the downloaded folder.
 - Follow the displayed step by step instructions. InstallShield installs the fitting software on your PC.
2. Select the preferred installation language: English, French, German or Dutch (for the installation procedure only) and click **OK**.
3. If additional Microsoft software components are required, they will be listed. Click **Install** to install them. If you are prompted to reboot, please follow the step-by-step instructions.
4. When the InstallShield Wizard appears, click **Next >**.
5. Read the license agreement and accept it to continue the installation process. Click **Next >**.
6. When the User Settings dialog appears, follow the instructions on the screen to select your country, your language and the installation type. Click **Next >**.
 - If NOAH 4 is installed, you can choose to install HANSATON scout under NOAH or Standalone, otherwise Standalone will be pre-selected.
7. Choose **Next >** on the Custom Setup screen to install all features in the default location, or follow the instructions to make changes.
8. Choose **Install** to begin the installation.
9. When prompted, select **Finish** to complete the installation.

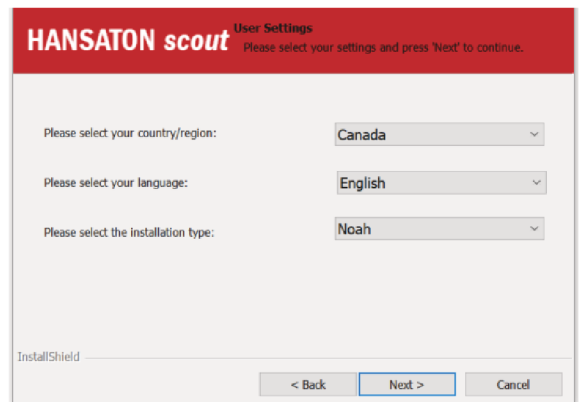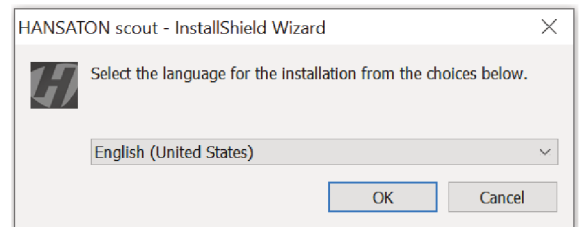System Requirements

Processor	Intel Core or higher performance
RAM	8 GB or more
Hard disk space	4 GB or more
Operating systems supported	> Windows 11, Home / Pro / Enterprise / Education > Windows 10, Home / Pro / Enterprise / Education
Display resolution	1280 x 1024 pixels
Patient secondary display resolution	1366 x 768 pixels
Graphic card	16 Million (24 bit) screen colors or more
Serial COM port	One for HI-PRO if used via serial COM port
USB ports	One for each purpose: > Bluetooth adaptor > Accessory programming > HI-PRO USB / HI-PRO 2 / iCube II / Noahlink Wireless / Noahlink Wireless 2
Programming interfaces	> iCube II > NOAHlink / Noahlink Wireless / Noahlink Wireless 2 > HI-PRO / HI-PRO USB / HI-PRO 2
NOAHlink driver	Latest version available
Noahlink Wireless driver	Latest version available
Internet connection	Highly recommended
Sound card	Stereo
Playback system	20 Hz - 14 kHz
Noah version	Latest version (Noah 4.4 or higher) Please check the Noah limitations for Windows operating systems on http://www.himsa.com
Automatic REM	Noah Version 4.4.2280 or higher Natus Otosuite 4.81.00 or higher Auditdata Primus version 4.1-5.2 Auditdata Measure version 6.0-6.4 Interacoustics Affinity Suite version 2.19-2.26 Signia Unity version 5.9-6.2

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REF 058-5237-057

sonova
HEAR THE WORLD

Type: PDL	Status: CURRENT
Document No.: PDL-4925	Effective Date: 20-Aug-2024
Rev.: 8	
Title: 029-6854-G4 Installation Guide - Hansaton scout v5.x software	
Process: Innovation Management	
Owner: 53DMUIR Destinee Muir	

Attributes

<u>Attribute Type</u>	<u>Value</u>	<u>Description</u>
Affected Site	5300	Unitron Hearing
Project	uPrince4 BTE	
Project	HANSATON scout 5.7.x	
Project Phase	Finished Product	

Approvals

<u>Level</u>	<u>Actor</u>	<u>Job Title</u>	<u>Sign-off Date</u>	<u>Sign-off By</u>
1	Bryan Hynes	Coordinator, Information Control	16-Aug-2024	53BHYNES
2	Zeljko Nadjmico	Head of Design Quality Assurance and Regulatory Affairs	19-Aug-2024	53ZNADJMICO
2	Betty Rule	Director HI Hansaton	20-Aug-2024	53BRULE
2	Jeff Hobbs	Manager, Engineering Master Data and Information Control	19-Aug-2024	53JHOBBS

Revision Notes

<u>Access Activity</u>	<u>Note</u>	<u>Accessed By</u>	<u>Accessed Date</u>
Remark	Updated for 5.7. Updated Automatic REM systems and added Noahlink Wireless 2 support. CHANGE-PLN-5300-01656	53DMUIR	16-Aug-2024

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Sonova AG
Sonova Deutschland GmbH